MAGNECISER™

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FOREWORD

This manual has been written for the owners and operators of the MAGNECISER™ resistive pedal exerciser. It contains general information on the operation, precautionary practices, safety, maintenance, and care instructions. In order to maximize use, efficiency and the life of your unit, please read this manual thoroughly and become familiar with the controls before operating the unit.

Specifications put forth in this manual were in effect at the time of publication. However, owing to Chattanooga Group's policy of continual improvement, changes to these specifications may be made at any time without obligation on the part of Chattanooga Group.

Before using the MAGNECISER™ resistive pedal exerciser, the users of this equipment should read, understand and follow the information contained in this manual for all unit functions (speed, time exercised, calories consumed, distance "traveled"), as well as the cautions ([pages 2-3](#)), and warnings ([page 3](#)). Consult other resources for additional information regarding pedal exercise therapy.

Product Description

The MAGNECISER™ resistive pedal exerciser is a portable, lightweight pedal exercise device designed to provide exercise therapy. The fully automatic 5-function digital display indicates speed, time exercised, calories consumed, distance "traveled" and scan of all functions.

The MAGNECISER™ resistive pedal exerciser also has a built-in adjustable base for stability and storage.

Keep informed on appropriate indications and contraindications for the use of pedal exercise therapy.

PRECAUTIONARY INSTRUCTIONS

The precautionary instructions found in this section and throughout this manual are indicated by specific symbols. Understand these symbols and their definitions before operating this equipment. The definition of these symbols are as follows:

 =**CAUTION**- Text with a "CAUTION" indicator explains possible safety infractions that have the potential to cause minor to moderate injury or damage to equipment.

 =**WARNING**- Text with a "WARNING" indicator explains possible Safety infractions that potentially cause serious injury and equipment damage.

 =**DANGER**- Text with a "DANGER" indicator explains possible safety infractions imminently hazardous situations that result in death or serious injury.

 =**EXPLOSION HAZARD**-

Text with an "Explosion Hazard" indicator explains possible safety infractions if this equipment is used in the presence of flammable anesthetics.

NOTE: Throughout this manual, "NOTE" may be found. These Notes are helpful information to aid in the particular area or function being described.

CAUTION

- Read, understand, and practice the precautionary and operating instructions. Know the limitations and hazards associated with using any pedal exercise device. Observe the precautionary and operational decals placed on the unit.
- Exercise of a strenuous nature, as is customarily done on this equipment, should not be undertaken without first consulting a physician. No specific health claims are made or implied as they relate to the equipment.
- Check unit before each use to determine all controls function normally.
- Handle unit with care. Inappropriate handling of the unit may adversely affect its characteristics.
- DO NOT permit any foreign materials or liquids to enter the unit. Take care to prevent any foreign materials including, but not limited to, inflammables, water, and metallic objects from entering the unit. These may cause unit damage, malfunction, or personal injury.
- DO NOT modify or remodel the unit. This may cause unit damage, malfunction, or personal injury.
- If you have difficulty operating the unit after carefully reviewing this operator's guide, contact your Chattanooga Group dealer for assistance.
- Failure to use and maintain the MAGNECISER™ pedal exercise unit in accordance with the instructions outlined in this manual will invalidate your warranty ([page 9](#)).

PRECAUTIONARY INSTRUCTIONS

CAUTION

- Know your heart rate and pulse, as well as your physician recommended target heart rate training zone. Monitor these during exercise.
- Proper medical clearance is recommended for anyone beginning an exercise program, especially if you are over 35 years of age or suffer from heart or respiratory problems.
- Warm up before any exercise programs with 8 minutes of aerobic activity.
- Wear comfortable clothes that allow freedom of movement and are not tight or restricting.
- Wear comfortable shoes with good support and nonslip soles.
- Breathe naturally, and never hold your breath during an exercise.
- Perform exercises consistently with proper technique, and with a full range of motion.
- Avoid over-training.
- After an exercise session, cool down with slow stretching, cycling, or walking.
- This unit should not be used by or near children.
- Handicapped or disabled people must have medical approval before using unit, and should be under close supervision when using any exercise equipment.
- Use this unit only for its intended purpose as described in this manual.
- This unit should only be used by one person at a time.
- Do not put hands, feet, or any foreign objects on or near this unit when in use by other people.
- Always operate this unit on a level surface.
- Never operate this unit if it is not functioning properly.
- Start exercise slowly and gradually increase the amount of resistance.
- If the user experiences dizziness, nausea, chest pain, or any other abnormal symptoms, stop the exercise at once and consult a physician immediately.
- Use caution to prevent fingers or hands from being pinched in moving parts when operating the unit.
- Do not attempt to carry the unit by the monitor ([page 6](#)).

WARNING

- Use of controls or adjustments or performance of procedures other than those specified herein may result in a hazardous exercise related injury.
- Dispose of all products in accordance with local and national regulations and codes.

SPECIFICATIONS

UNIT SPECIFICATIONS

Dimensions*

Width	39 cm (15.5 inches)
Length	48 cm (19 inches)
Height	46 cm (18 inches)

Weight

Standard Weight.....	8 kg (18 lbs)
Shipping Weight.....	10 kg (22 lbs)

Display

Alpha numeric display directs choice of display modes. Display includes functions of time, speed, calories burned, and distance.

Approvals


Unit Constructed in compliance with UL 2601-1, CSAC22.2 No. 601.1, and IEC 601 Standards. Complies with Directive 93/42/EEC.

Power

Voltage	two 1.5 V "AA" batteries
Electrical Class and Type	CLASS I, TYPE B

*Measurements are approximate.

NOMENCLATURE

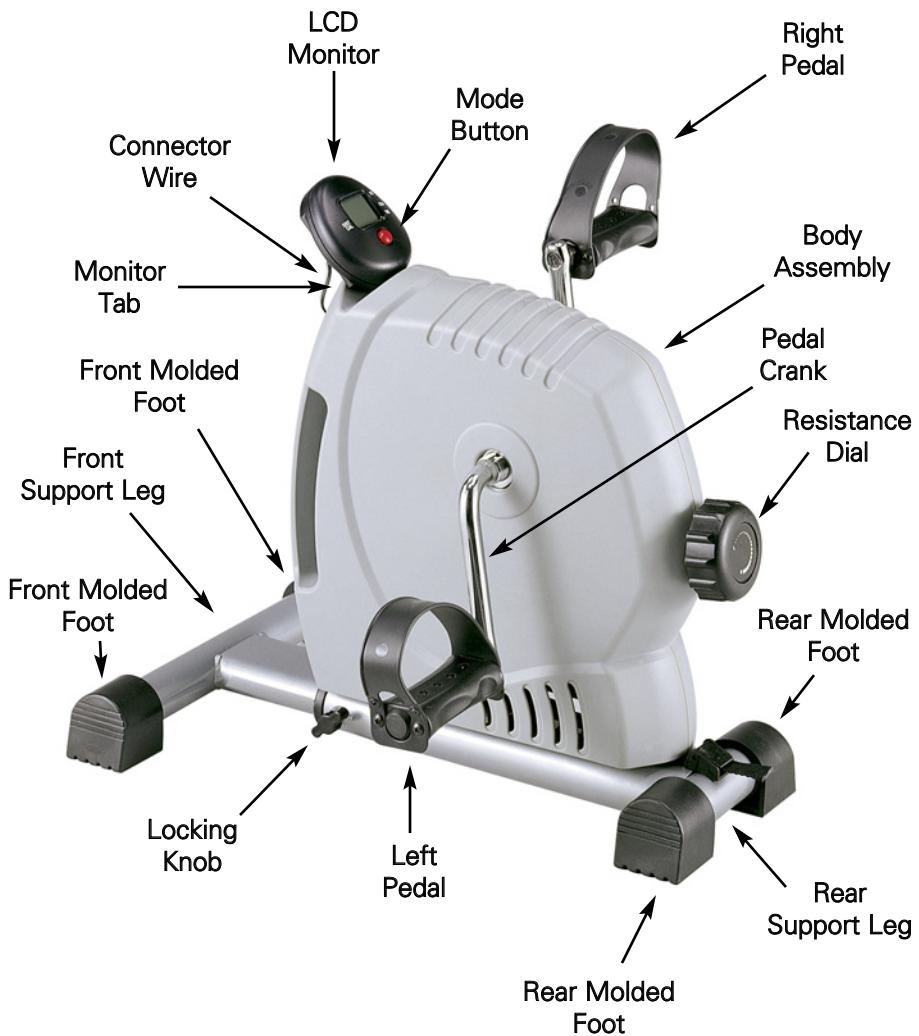NOMENCLATURE

LCD Monitor

The LCD (Liquid Crystal Display) Monitor is an alphanumeric display that allows the user to view the speed, time exercised, calories consumed, distance "traveled" and scan of all functions. The LCD Monitor will automatically begin displaying information when you start exercising, or if you press the Mode Button. The LCD Monitor automatically shuts off if no input signal is received within a 4-5 minute period. The LCD Monitor requires two 1.5v "AA" batteries that are included in the shipping.

NOTE: Do not attempt to carry the unit by the monitor.

Mode Button

Press the Mode Button to select functions of time, speed, calories burned, and distance. Specifically, the Mode button selects the following:

DIST: accumulates the workout distance when starting the exercise.


CAL: calories you burn during your workout time.

SPEED: current speed (in Miles Per Hour) travelled during workout time.

SCAN: automatically scans through all the functions, and displays each briefly on the LCD Monitor.

TIME: accumulates the total duration of workout time.

Reset: Hold down the Mode Button for two seconds in order to reset all functions.

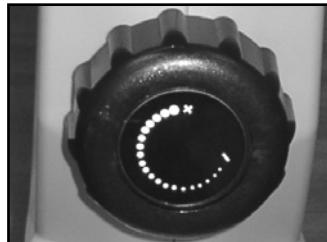Resistance Dial

The Resistance Dial controls the amount of resistance applied during exercise. Turn the Tension Dial clockwise for a more difficult workout, and counter clockwise for a less difficult workout.

SETUP

ASSEMBLY

To assemble the MAGNECISER™ resistive pedal exerciser, do the following:

1. On a flat surface, position the Body Assembly so that the Resistance Dial is facing you.

2. Push the Rear Molded Feet onto the Rear Support Leg.

3. Push the Front Molded Feet onto the Front Support Leg.

4. Insert the Rear Support Leg into the Body Assembly.

Note: Keep in mind that the length can be adjusted later before operation.

SETUP

ASSEMBLY (continued)


5. Insert the Locking Knob into the Body Assembly where appropriate, and turn it clockwise until it is snug (where the Front Support Leg does not slide out of the Body Assembly).

CAUTION: Do not over-tighten the Locking Knob.


6. Insert the threads of the Right Pedal into the Right Pedal Crank, and with the wrench, turn it counter clockwise until it is snug. Repeat this process for the Left Pedal.

CAUTION: Do not over-tighten the pedals.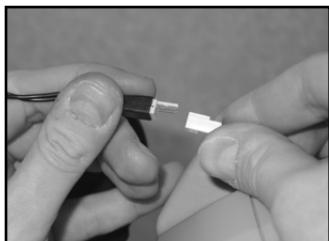

7. Plug the wire from the Monitor into Connector Wire.

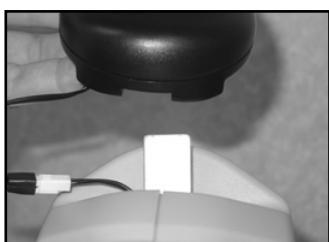8. Slide the Monitor completely onto the Monitor Tab.

MAINTENANCE & WARRANTY

Cleaning the MagneCiser™ Resistive Pedal Exerciser

The unit can be cleaned with a damp cloth. Do not use abrasive cleansers. Prevent liquid and foreign materials from entering the unit. Read and follow the Precautionary Instructions ([page 2](#)), Cautions ([pages 2-3](#)), and Warnings ([page 3](#)).

Warranty

Chattanooga Group ("Company") warrants that the MagneCiser™ resistive pedal exerciser is free of defects in material and workmanship. This warranty shall remain in effect for one year (12 months) from the date of original consumer purchase. If this Product fails to function during the one year warranty period due to a defect in material or workmanship, Company or the selling dealer will repair or replace this Product without charge within a period of thirty (30) days from the date on which the Product is returned to the Company or the dealer.

All repairs to the Product must be performed by a service center certified by the Company. Any modifications or repairs performed by unauthorized centers or groups will void this warranty.

To participate in warranty coverage, this Product's warranty registration card (included with Product) must be filled out and returned to the Company by the original owner within ten (10) business days of purchase.

This Warranty Does Not Cover:

Replacement parts or labor furnished by anyone other than the Company, the selling dealer or a certified Company service technician.

Defects or damage caused by labor furnished by someone other than Company, the selling dealer or a certified Company service technician.

Any malfunction or failure in the Product caused by product misuse, including, but not limited to, the failure to provide reasonable and necessary maintenance or any use that is inconsistent with the Product User's Manual.

COMPANY SHALL NOT BE LIABLE IN ANY EVENT FOR INCIDENTAL OR CONSEQUENTIAL DAMAGES.

Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

To Obtain Service From Company or the selling dealer under this warranty:

1. A written claim must be made within the warranty period to the Company or the selling dealer. Outside of the U.S.A., written claims made to the Company should be sent to:

Chattanooga Group
4717 Adams Road
P.O. Box 489
Hixson, TN 37343 USA
Telephone: +1 423 870-7200
Facsimile: +1 423 870-2046
www.chattgroup.com

and

2. The Product must be returned to the Company or the selling dealer by the owner.

This warranty gives you specific legal rights and you may also have other rights which vary from state to state.

The Company does not authorize any person or representative to create for it any other obligation or liability in connection with the sale of the Product. Any representation or agreement not contained in the warranty shall be void and of no effect.

THE FOREGOING WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING ANY WARRANTY OR MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.

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More Trusted Products from Chattanooga Group

Achiever™
Supports

Adapta®
Treatment Tables

A.E.R.® Boot and Compression System
Auto Edema Reduction Boot

Baseline® Measurement Instruments
Dynamometers, Goniometers, etc.

boo-boo pac™
Child Size Bear-Shaped Cold Pack

Cambion®
Shock Dampening Foot Care Products

Carpal-Trac™
Carpal Traction Accessory

Cervical Traction System
Clinical Cervical Traction

Chattanooga Exerciser™
Exercise Equipment

ColPaC®
Chilling Units and Reusable Cold Therapy Products

Conductor™ Gel
Highly Conductive Ultrasound Gel

Contracture Products
Contracture Management Orthotic Products

CTS®
Carpal Tunnel Stretching Device

DURA-STICK® Electrodes
Self-Adhesive Electrodes

EMG Retrainer® & EMG Retrainer® IR
Dual Channel Surface EMG

Flexi-PAC®
Reusable Hot and Cold Compresses

Fluidotherapy®
Dry Heat Therapy Units

Forte™ CPS
Electrotherapy and Ultrasound

Gel Medex™
Gel Mattress Overlay

Hydrocollator®
Heating Units and HotPacs™

Intelect® Legend
Ultrasound and Electrotherapy Products

Magneciser™
Exercise Equipment

Myossage®
Massage Lotion

Nylatex®
Elastic Wraps

OptiFlex® and OptiFlex® S
Continuous Passive Motion

Opti-Ice™
Cold Therapy System

Para-Care®
Paraffin Wax Unit and Accessories

Pillow Perfect™
Cervical Pillow Line

Pivotal Therapy System™
Orthotics for the Spine

ProPower Pillow™
Power Massage Pillow

PresSsion®
Intermittent Compression

Pron Pillo®
Positioning Pillow

QuikWrap™
Deluxe Universal Belt System

SensaFlex™
Hot and Cold Compress

SPORT-PAC™
Soccer Ball Shaped Cold Pack

Theratherm™
Digital Moist Heating Pad

Therma-Wrap™
Hot and Cold Compression

Triton®
Treatment and Traction Equipment

Tru-Trac®
Traction Equipment

TX®
Treatment and Traction Equipment

Vectra™ Series
Electrotherapy and Ultrasound

Versa Bath Seat™
Aid to Daily Living

Wellness 1st™
Back Support

Women's Contour Back Support
Back Support

Bringing
Innovation to
the Rehab
World!™

ISO 9001/ISO 13485 Certified

4717 Adams Road
P.O. Box 489
Hixson, TN 37343 U.S.A.
1-423-870-2281
1-800-592-7329 U.S.A.
1-800-361-6661 CANADA
+1 423-870-2046 Outside USA FAX
www.chattpgroup.com

© 2004 Encore Medical